

# A VOICE...

# WHEN YOURS CAN'T BE THERE



*Living with the risk of anaphylaxis...*

*We'll talk you through it!* **Allerject**™

*Allerject™ Voice-Assisted Epinephrine Auto-Injector*

*Available in two dosage strengths*



**Allerject™ 0.15 mg** for patients weighing **15 kg to 30 kg (33 lb. to 66 lb.)**  
(For children under 15 kg, talk to your doctor about using other forms of injectable epinephrine.)

Available in two languages:  
English or French

Unidose 0.15 mg/0.15 mL  
epinephrine auto-injector [1:1000]



**Allerject™ 0.3 mg** for patients weighing **over 30 kg (66 lb.)**

Available in two languages:  
English or French

Unidose 0.3 mg/0.3 mL  
epinephrine auto-injector [1:1000]

Allerject™ voice-assisted auto-injectors are indicated for the emergency treatment of anaphylactic reactions in patients who are determined to be at increased risk for anaphylaxis.

To receive your  
Allerject™ Trainer,  
please order on line  
at [www.allerject.ca](http://www.allerject.ca)

For more information concerning  
Allerject™ call 1-855-405-4321



## HOW TO USE ALLERJECT™

Practice With Allerject™ Trainer First

Allerject™ is the first and only epinephrine auto-injector that will talk users through each step of the injection process. Allerject™ was designed to be easy to use. It has audio instructions and visual cues/lights up when the injection is complete, as well as a retractable needle. Alternatively, a patient or caregiver can follow the written instructions printed on the device. If the voice instructions do not work for any reason, use Allerject™ as instructed in the following steps. It will still work during an allergic reaction emergency.



### 1. Pull Allerject™ from the outer case.

**Do not** go to step 2 until you are ready to use Allerject™. If you are not ready to use, put it back in the outer case.



### 2. Pull off RED safety guard.

To reduce the chance of an accidental injection, do not touch the black base of the auto-injector, which is where the needle comes out. If an accidental injection happens, get medical help immediately.

**NOTE:** The safety guard is meant to be tight. **Pull firmly to remove.**

### 3. Place **BLACK** end **AGAINST** the **MIDDLE** of the **OUTER** thigh (through clothing, if necessary), then **press firmly and hold in place for five seconds.**



Only inject into the middle of the outer thigh (upper leg). Do not inject into any other location.

**NOTE:** Allerject™ makes a distinct sound (click and hiss) when you press it against your leg. This is normal and indicates Allerject™ is working correctly.

### 4. Seek immediate medical or hospital care.

Replace the outer case and take your used Allerject™ with you to your pharmacist or physician for proper disposal and replacement.

### **AFTER** using Allerject™ seek immediate medical attention

Please consult the Consumer Information leaflet in your product package for complete dosage and administration instructions.

With a severe, long-lasting allergic reaction, you may need to administer an additional dose of epinephrine. More than two sequential doses of epinephrine should only be administered under direct medical supervision.

It is important that you seek immediate medical assistance or go to the emergency room immediately after using Allerject™. Following treatment, the patient must stay within close proximity to a hospital or where they can call 911 for the next 48 hours. To ensure Allerject™ is right for you, always read and follow the label.

## HOW TO STORE ALLERJECT™

- Keep your Allerject™ at room temperature. Do not refrigerate. Do not drop.
- Occasionally inspect your Allerject™ solution through the viewing window. Replace your Allerject™ if it is discolored or contains solid particles (precipitate) or if there are any signs of leakage. The solution should be clear.